

skin affix™ *Application Guide*

STEP ONE: ACTIVATE DEVICE




While holding upright, squeeze middle of the SkinAffix™ device until a crackling sound is heard.

STEP TWO: PRIME DEVICE



Invert the tip and gently squeeze up to three times to prime the device. When the tip turns purple, SkinAffix™ is ready to use. Ensure that wound is thoroughly clean and dry prior to closure.

STEP THREE: APPLY ADHESIVE

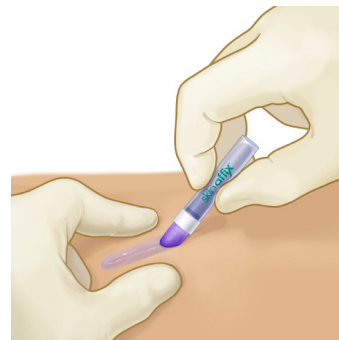

While holding the wound edges together, apply a single, even coat of SkinAffix™ over the length of the wound, using the tip of the device to distribute and prevent pooling of the adhesive.

Note that applying excessive adhesive may lead to a delay in dry time.

Continue to hold the wound edges together for approximately 60 seconds. Do not apply dressings or gowns until SkinAffix™ is completely dry (approx. 2mins).