


Instructions

Exuderm LP

Exuderm LP is a wound dressing called a hydrocolloid. It helps absorb light amounts of wound fluid while providing a waterproof, protective barrier.

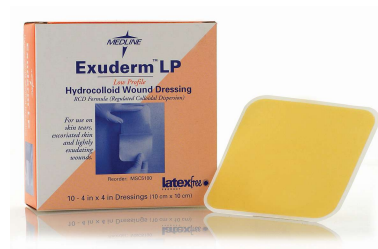

Applying Exuderm LP

- Cleanse wound with normal saline or wound cleanser and gently dry surrounding area.
- Dressing should extend 1/2" beyond the edges of the wound.
- Gently stretch the dressing horizontally to break release liner. Remove one side and place adhesive portion on the skin.
- Remove the other liner and place remaining half of adhesive portion on the skin.
- Place hand on dressing and hold for one minute to warm dressing and secure adhesion.

When to change / reapply Exuderm LP

- Dressing should be left in place for the longest possible time, up to 7 days, or as directed.
- If fluid leaks from dressing, change it immediately.

Removing Exuderm LP

- Press down on the skin and carefully lift the dressing starting at the edge and pulling toward the center.

What to expect

- As the dressing absorbs wound fluid, it may look "melted." This may appear slimy, but is normal. It can be easily cleansed away.

When to call your health care provider

- If you experience a sudden change in your condition such as: increased pain, bleeding that won't stop, a fever, or sudden swelling, contact your health care provider.

Additional Instructions

Exuderm LP Ordering Details

<u>Item No.</u>	<u>Description</u>	<u>Packaging</u>
MSC5100	4" x 4"	10/bx
MSC5125	6" x 6"	5/bx

See product insert for complete safety and usage information.