

# Instructions

## Exuderm Ultra

Exuderm Ultra is a wound dressing called a hydrocolloid. It helps absorb wound fluid while providing a protective barrier.



### Applying Exuderm Ultra

- Cleanse wound with normal saline or wound cleanser and gently dry surrounding area.
- Dressing should extend 1/2" beyond the wound margin.
- Gently stretch the dressing horizontally to break release liner. Remove one side and place adhesive portion on the skin.
- Remove other liner and place remaining adhesive portion on the skin.
- Place hand on dressing and hold for 1 minute to warm dressing and finalize adhesion.

### When to change / reapply Exuderm Ultra

- The dressing should be left in place for the longest possible time, up to 7 days, or as directed.
- If fluid leaks from dressing, change it immediately.

### Removing Exuderm Ultra

- Press down on the skin and carefully lift the dressing starting at the edge and pulling toward the center.

### What to expect

- If the dressing is used in an area that is covered with an incontinence product (such as an adult diaper or brief), it may need to be changed more frequently.

### When to call your health care provider

- If you experience a sudden change in your condition such as: increased pain, bleeding that won't stop, a fever, or sudden swelling, contact your health care provider.

### Additional Instructions

#### Exuderm Ultra Ordering Details

<u>Item No.</u>	<u>Description</u>	<u>Packaging</u>
MSC5300	4" x 4"	10/bx

**See product insert for complete safety and usage information.**