

Optifoam Ag

Superb fluid handling plus antimicrobial protection



Applying Optifoam Ag

- Cleanse the wound with sterile saline or appropriate wound cleanser such as Skintegrity Wound Cleanser.
- Dry the surround skin.
- Select the appropriate size dressing to allow the foam pad to cover all breached or compromised skin.
- For the adhesive version, remove the outer most liner, center the dressing over the wound and adhere the exposed edge and remove the remaining liner, smoothing the entire dressing over the wound.
- For the non-adhesive version, apply the white foam side against the wound (pink side UP and AWAY from the patient). Secure with tape such as Medfix non-woven tape or elastic net or Medigrip tubular elastic bandage if adhesive-free securement is preferred.

When to change / reapply Optifoam Ag

- Dressings can be left in place for up to 7 days. Dressing change is indicated if leakage or strikethrough occurs.

Removing Optifoam Ag

- Remove the outer securement dressing or tape gently and remove the Optifoam dressing.

What to expect

- The dressing will absorb the exudate and its characteristic color and odor.

When to call your health care provider

- If you experience a sudden change in your condition such as: increased pain, bleeding that won't stop, a fever, or sudden swelling, contact your health care provider.

Additional Instructions

Optifoam Ag Ordering Details

<u>Item No.</u>	<u>Description</u>	<u>Packaging</u>
MSC9604	Optifoam Ag Adhesive 4" x 4" (2.5 x 2.5	10/bx, 100/cs
MSC9614	Optifoam Ag Non-Adhesive 4" x 4"	10/bx, 100/cs