


Instructions

SilvaSorb Cavity

SilvaSorb Cavity dressings have the ability to both absorb wound fluid and help provide moisture, depending on conditions in the wound. SilvaSorb Cavity is designed for packing deeper wounds. They also contain microscopic particles of controlled-release ionic silver to help provide antimicrobial protection.

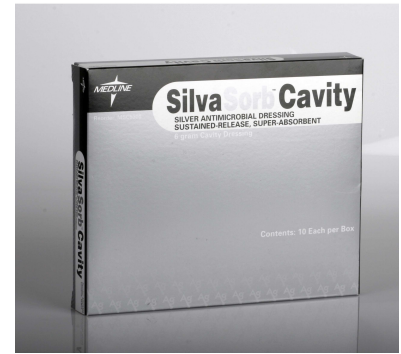

Applying SilvaSorb Cavity

- Cleanse wound with normal saline or wound cleanser and gently dry surrounding area.
- Remove the blue liner. If the dressing is too large for the wound, then pull off some of the strands to make a smaller dressing.
- Since SilvaSorb Cavity swells as it absorbs moisture, the dressing should only fill up to one-half of the wound depth.

When to change / reapply SilvaSorb Cavity

- Dressing may be left in place for up to seven days, or as directed by your health care professional.
- If fluid from the wound is leaking from the dressing, the dressing should be changed immediately.

Removing SilvaSorb Cavity

- Carefully remove the secondary dressing and SilvaSorb Cavity from the wound.
- SilvaSorb Cavity can be remoistened with saline or wound cleanser to ease removal.

What to expect

- As SilvaSorb Cavity absorbs wound fluid, it increases dramatically in size. It also may absorb the color and odor of the wound.

When to call your health care provider

- If you experience a sudden change in your condition such as: increased pain, bleeding that won't stop, a fever, or sudden swelling, contact your health care provider.

Additional Instructions

SilvaSorb Cavity Ordering Details

<u>Item No.</u>	<u>Description</u>	<u>Packaging</u>
MSC9360	Cavity, 6 gram	5/bx, 5 bx/cs

See product insert for complete safety and usage information.