

Instructions

SilvaSorb Gel

SilvaSorb Gel helps hydrate dry wounds and helps provide antimicrobial protection with microscopic particles of controlled-release ionic silver.



Applying SilvaSorb Gel

- Cleanse wound with normal saline or wound cleanser and gently dry surrounding area.
- Apply SilvaSorb Gel approximately 1/8 - 1/4" thick to wound area.
- Cover wound with appropriate waterproof cover dressing, such as Stratasorb.

When to change / reapply SilvaSorb Gel

- Dressing may be left in place up to 3 days, or as directed.

Removing SilvaSorb Gel

- Carefully remove the secondary dressing.
- The gel may be removed with the cover dressing.
- If the gel dries, rinse with normal saline or wound cleanser.

What to expect

- SilvaSorb gel may become discolored as it mixes with fluid from the wound.

When to call your health care provider

- If you experience a sudden change in your condition such as: increased pain, bleeding that won't stop, a fever, or sudden swelling, contact your health care provider.

Additional Instructions

SilvaSorb Gel Ordering Details

<u>Item No.</u>	<u>Description</u>	<u>Packaging</u>
MSC9301	Gel, 1.5 oz tube	12/cs
MSC93025	Gel, .25 oz tube	25/bx
MSC9308	Gel, 8 oz tube	6/cs
MSC9316	Gel, 16 oz net. wt. jar	8/cs

See product insert for complete safety and usage information.