

Instructions

SilvaSorb Sheets

SilvaSorb Sheets have the ability to both absorb wound fluid and help provide moisture, depending on conditions in the wound. They also contain microscopic particles of controlled-release ionic silver to help provide antimicrobial protection.



Applying SilvaSorb Sheets

- Cleanse wound with normal saline or wound cleanser and gently dry surrounding area.
- Place either side of dressing on the wound. The dressing should extend 1/2" beyond the wound edges or SilvaSorb may be cut to size.
- Secure with an appropriate dressing like a transparent film or a composite island dressing or simply secure with tape.

When to change / reapply SilvaSorb Sheets

- Dressing may be left in place up to 7 days, or as directed.
- If fluid leaks from the dressing, change it immediately.

Removing SilvaSorb Sheets

- Carefully remove the secondary dressing and SilvaSorb from the wound.
- SilvaSorb can be remoistened with saline or wound cleanser to ease removal.

What to expect

- As dressing absorbs fluid from the wound, it may become discolored and slimy.

When to call your health care provider

- If you experience a sudden change in your condition such as: increased pain, bleeding that won't stop, a fever, or sudden swelling, contact your health care provider.

Additional Instructions

SilvaSorb Sheets Ordering Details

| <u>Item No.</u> | <u>Description</u> | <u>Packaging</u> |
|-----------------|------------------------------|------------------|
| MSC9322 | Sheet, 2" x 2" | 5/bx, 5 bx/cs |
| MSC9340 | Perf. Sheet, 4-1/4" x 4-1/4" | 5/bx, 5 bx/cs |
| MSC93410 | Perf. Sheet, 4" x 10" | 5/bx, 5 bx/cs |
| MSC9344 | Sheet, 4-1/4" x 4-1/4" | 5/bx, 5 bx/cs |
| MSC9348 | Sheet, 4" x 8" | 5/bx, 5 bx/cs |

See product insert for complete safety and usage information.