

Instructions

SilvaSorb Site

SilvaSorb Site is dressing designed to fit around I.V. catheters, venous lines and other port, pin or catheter site. The dressing absorbs fluid and contains microscopic particles of controlled-release ionic silver to help provide antimicrobial protection.



Applying SilvaSorb Site

- Cleanse wound with normal saline or wound cleanser and gently dry surrounding area.
- Remove the blue backing from the dressing.
- Gently wrap the dressing around the line site or device, placing either side of the dressing down against the skin. The two sides of the slit can then be brought back together and overlapped.
- Secure the catheter and SilvaSorb Site dressing to the skin with a transparent film dressing.

When to change / reapply SilvaSorb Site

- Dressing may be left in place up to 7 days, or as directed.
- If fluid leaks from the dressing, change it immediately.

Removing SilvaSorb Site

- To remove the transparent film dressing, lift one edge and stretch the film laterally to the skin surface, while holding the catheter securely in place. The SilvaSorb Site dressing should lift away along with the film dressing.

When to call your health care provider

- If you experience a sudden change in your condition such as: increased pain, bleeding that won't stop, a fever, or sudden swelling, contact your health care provider.

Additional Instructions

SilvaSorb Site Ordering Details

<u>Item No.</u>	<u>Description</u>	<u>Packaging</u>
MSC9310	Site, 1" circular pad w/slit	10/bx, 6 bx/cs
MSC9320	Site, 1-3/4" circular pad w/slit	30/cs

See product insert for complete safety and usage information.