

Instructions

Skintegrity Hydrogel

Skintegrity Hydrogel is a clean, greaseless hydrogel that helps keep the wound hydrated.



Applying Skintegrity Hydrogel

- Cleanse wound with normal saline or wound cleanser and gently dry surrounding area.
- Apply Skintegrity Hydrogel approximately 1/8-1/4" thick to wound area.
- Cover wound with appropriate waterproof cover dressing, such as Stratasorb. Skintegrity hydrogel may dry out sooner if a non-waterproof dressing (such as gauze) is used.

When to call your health care provider

- If you experience a sudden change in your condition such as: increased pain, bleeding that won't stop, a fever, or sudden swelling, contact your health care provider.

When to change / reapply Skintegrity Hydrogel

- Dressing may be left in place up to 3 days, or as directed.

Additional Instructions

Skintegrity Hydrogel Ordering Details

<u>Item No.</u>	<u>Description</u>	<u>Packaging</u>
MSC6102	Bellows Bottle, 1 oz.	30/cs
MSC6104	Tube, 4 oz.	12/cs

See product insert for complete safety and usage information.