

# Instructions

## Skintegrity Hydrogel Gauze

Skintegrity Hydrogel Gauze is a gauze dressing that is coated with hydrogel to help pack deep or tunneling wounds and keep the wound hydrated.



### Applying Skintegrity Hydrogel Gauze

- Cleanse wound with normal saline or wound cleanser and gently dry surrounding area.
- Fill wounds loosely with unfolded, saturated gauze pads.
- Cover wound with appropriate waterproof cover dressing, such as Stratasorb. Skintegrity hydrogel may dry out sooner if a non-waterproof dressing is used.

### When to call your health care provider

- If you experience a sudden change in your condition such as: increased pain, bleeding that won't stop, a fever, or sudden swelling, contact your health care provider.

### When to change / reapply Skintegrity Hydrogel Gauze

- Dressing may be left in place up to 3 days, or as directed.
- If fluid leaks from the dressing, change it immediately.

### Additional Instructions

### Skintegrity Hydrogel Gauze Ordering Details

<u>Item No.</u>	<u>Description</u>	<u>Packaging</u>
MSC6022	2" x 2", 12-ply	1/pk, 50/cs
MSC6044	4" x 4", 12-ply	1/pk, 30/cs
MSC6144	4" x 4", 12-ply	2/pk, 30 pk/cs

**See product insert for complete safety and usage information.**