


Instructions

Skintegrity Hydrogel Sheet

A moist, soothing, cooling, water-rich hydrogel sheet

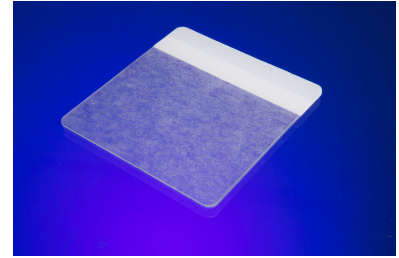

Applying Skintegrity Hydrogel Sheet

- Cleanse wound with sterile saline or appropriate wound cleanser such as Skintegrity Wound Cleanser.
- Remove dressing from pouch, cut the dressing to desired size if necessary.
- Apply a skin protectant such as Sureprep No-Sting preps to periwound skin to help prevent maceration. Allow to dry completely.
- Remove the THICK plastic protective film and apply the exposed side of the gel to the wound.
- Apply gentle pressure around the dressing perimeter to ensure the dressing is secure.
- Secure the dressing with a suitable tape or with an appropriate secondary dressing such as StrataSorb or Bordered Gauze.
-

Removing Skintegrity Hydrogel Sheet

- Gently remove the securement tape or dressing and remove the Skintegrity Gel Sheet.

When to call your health care provider

- If you experience a sudden change in your condition such as: increased pain, bleeding that won't stop, a fever, or sudden swelling, contact your health care provider.

When to change / reapply Skintegrity Hydrogel Sheet

- The dressing may be left in place up to 72 hours

Additional Instructions

Skintegrity Hydrogel Sheet Ordering Details

<u>Item No.</u>	<u>Description</u>	<u>Packaging</u>
MSC6268	Skintegrity Hydrogel Sheet 6" x 8"	5/bx, 20 bx/cs

See product insert for complete safety and usage information.