

FRONT

GUARDIAN[®]
Family-owned. Trusted by hospitals for over 50 years.

**BARIATRIC QUAD CANE,
LARGE BASE**

CANNE DE TYPE QUAD
POUR BARIATRIE, GRANDE BASE

BASTÓN 4 PATAS
BARIATRICO, BASE GRANDE

- Hospital Quality
- Height Adjustable
- Locking Collar for Safety
- Holds up to 500 lbs. (227 kg)

1 QUAD CANE
Black/Noir/Negro

REF MDS86228XW

www.medline.com ©2016 Medline Industries, Inc.
Manufactured in China for/Fabriqué en Chine pour/
Fabricado en China para: Medline Industries, Inc.
Northfield, IL 60093 USA 1-800-MEDLINE RJ16RMM
Guardian and Medline are registered trademarks
of Medline Industries, Inc.

This Product Contains Dry Natural Rubber.
Ce produit contient du caoutchouc naturel sec.
Este producto contiene caucho natural seco.

LIFETIME
LIMITED
WARRANTY

GARANTIE
LIMITÉE À LA
DURÉE DE VIE

GARANTÍA
LIMITADA
DE POR VIDA

BACK

Instructions for Adjustable Quad Cane

This offset handle cane with foam grip is designed for maximum comfort and support.

To adjust height:

1. Loosen locking ring by rotating counter clockwise.
2. Depress the push button and slide the leg to the desired height, up or down, confirm the push button is fully engaged in the desired hole.
3. Tighten the locking ring by rotating the knob clockwise until completely secure.

To adjust for right or left hand use:

1. Loosen locking ring by rotating counter clockwise.
2. Depress the push button and rotate handle 180 degrees, confirm push button locks into desired height hole on other side of cane.
3. Tighten the locking ring by rotating clockwise until completely secure.
4. Re-adjust height as needed.

PMS COLORS USED		● CMYK	
Black	541		

6.96x11.43 cm