LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2018 Medline Industries, Inc. Guardian and Medline are registered trademarks of Medline Industries, Inc.

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GUARDIAN

Family-owned. Trusted by hospitals for over 50 years.

LOCKING RAISED TOILET SEAT USER INSTRUCTIONS & WARRANTY

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REF G30260

REF G30270A

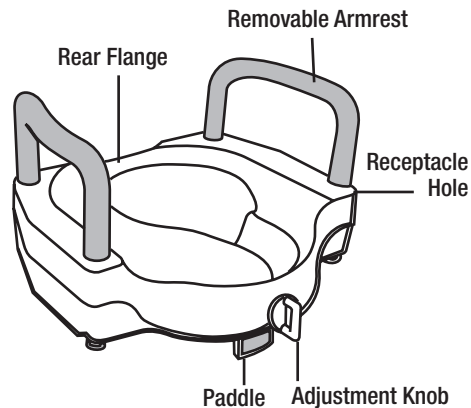
Please take a moment to read this instruction sheet to familiarize yourself with the safe and correct way to use your new locking raised toilet seat.

WARNINGS:

- ⚠ **DO NOT** assemble or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment. Otherwise, injury or damage may occur.
- ✓ Certain toilet fixtures will prevent the locking raised toilet seat from working properly and or will damage the locking raised toilet seat. If the toilet bowl has on its top or inner ring any raised or protruding points or sections, the locking raised toilet seat should not be used.
- ✓ **BEFORE** each use, ensure that ALL attaching hardware is securely tightened.
- ✓ Use the arm supports **ONLY** for assistance. **“DO NOT”** attempt to use the arms to support full body weight.
- ✓ Always check to make sure that the raised toilet seat is properly and securely locked in place before using. Make certain that the knob that secures the raised toilet seat to the commode is tight at all times.
- ✓ Center your weight on the raised toilet seat. The seat may tip if you lean too far forward to either side, or if equipped by leaning on only one armrest.
- ⚠ Users with limited physical capabilities should be supervised or assisted when using the raised toilet seat.
- ✓ The raised toilet seat’s weight capacity is: 350 lbs. (159 kg.)
- ✓ Rotate the adjustment knob by hand only to prevent over-torque of the assembly. Use of a wrench or other tool will break the screw assembly. **DO NOT FORCE** knob as this may damage the knob and/or the raised toilet seat.
- ✓ The Medline raised toilet seat is not designed for permanent installation. The unit should be loosened and retightened at least once a week to ensure a proper fit.
- ⚠ If, for any reason, a proper fit in installation cannot be achieved, **“DO NOT”** use and contact retailer where purchased for return instructions.
- ✓ Use the arm supports **ONLY** for assistance. During use, weight should be evenly distributed between the arm rests. **DO NOT** attempt to use the arms to support full body weight.

ASSEMBLY INSTRUCTIONS (MODEL G30270A ONLY)

With the detent buttons towards the front, insert the armrest tubing into the receptacle holes on the raised toilet seat until the detent button engages.

MODEL G30270A WITH ARMS

INSTALLATION INSTRUCTIONS

The Guardian raised toilet seat has been designed to accommodate a wide variety of toilet bowl types. However, certain bowl types will require the removal of the existing seat and/or lid to ensure proper fit when installed.

1. Lift the seat and lid of the toilet or remove.
2. Rotate adjustment knob counter-clockwise until the paddle is far enough back to clear the inner rim of the bowl.
3. Place the raised toilet seat onto the bowl, hooking rear flanges under edge of bowl. Check that raised seat rests flat on bowl rim. The knob must be at the front end of the toilet bowl.
4. Center raised seat on bowl rim and push it back as far as possible with your hands.
5. Rotate adjustment knob clockwise to tighten. Turn until resistance is felt, then turn an additional 1/8” turn until snug.
CAUTION— Rotate the adjustment knob by hand only to prevent over-torque of that assembly. Use of a wrench or other tool will break the screw assembly.
6. If the top rear of raised seat is hitting the toilet seat or lid, remove the toilet seat and lid and repeat steps 2 to 5.

WARNING

The Guardian raised toilet seat is not designed for permanent installation. The unit should be loosened and retightened once a week to ensure a proper fit.

When the raised toilet seat is properly installed, the unit will sit flat on the top of the lip of the bowl and should fit securely against the inner rim of the bowl.

USER INSTRUCTIONS

After proper installation of the unit, follow these instructions carefully. Failure to comply can result in severe injury. The Guardian raised toilet seat may be used in the same manner as a regular toilet seat.

1. Back up to the raised toilet seat until you feel the back of your knees touching it.
2. Reach back for the armrests (model G30270A). Evenly distribute weight between the armrests. **DO NOT** fully support yourself with one armrest. Use the arm supports **ONLY** for assistance. Weight should be evenly distributed between the armrests. **DO NOT** attempt to use the arms to support full body weight.
3. Slowly lower yourself onto the raised toilet seat by bending your elbows and, if possible, your hips and knees.
4. Reverse Steps 1 through 3 for getting up. Push on the armrests or seat as much as is comfortably possible. Make sure you have your balance before releasing your grasp from the raised toilet seat.

CLEANING INSTRUCTIONS

This raised toilet seat can be easily cleaned with regular household cleaning products. **DO NOT** use abrasive powders or scouring pads.

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