LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2018 Medline Industries, Inc. Guardian and Medline are registered trademarks of Medline Industries, Inc.

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GUARDIAN

Family-owned. Trusted by
hospitals for over 50 years.

TOILET SAFETY FRAME USER INSTRUCTIONS & WARRANTY

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REF G30300-1

IMPORTANT SAFEGUARDS

Read all instructions before using.

Important information is highlighted by these terms:

WARNING— Important safety information for hazards that might cause serious injury.

CAUTION— Information for preventing damage to the product.

NOTE— Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

Please read carefully before use. Failure to comply may result in severe injury.

Do not exceed the 300 lb. maximum weight capacity of this safety frame.

Evenly distribute weight between the armrests during use.
Ensure all detent buttons are properly engaged in position prior to use.

FAILURE TO FOLLOW THESE INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

SAVE THESE INSTRUCTIONS.

ASSEMBLY INSTRUCTIONS

Before beginning assembly, identify all parts.

1. Remove existing toilet seat / lid assembly.
2. Place toilet safety frame bracket between toilet fixture and toilet seat / lid assembly with rounded part of bracket facing back of toilet (Figure 1).
3. Replace toilet seat / lid and reinsert mounting bolts through bracket and toilet fixture. Replace and tighten nuts.
4. Insert side frames into bracket. Depress detent button and slide side frame into desired position (Figure 2).
5. Adjust footpieces to desired height. Make sure both footpieces are fully engaged and adjusted at equal heights.
6. Make sure mounting hardware is tightened and all detent buttons are fully engaged before use.

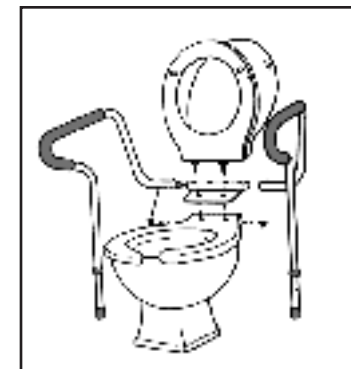
Figure 1

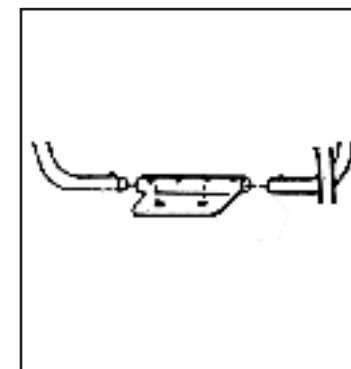
Figure 2

CLEANING

Your safety frame can be easily cleaned with regular household cleaning products. DO NOT use abrasive powders or scouring pads.

Grip Cleaning Instructions

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

***Tips on This Product Contain
Dry Natural Rubber.**

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