



**Dial
Floor Scale**
Instructions for Use

REF MDR300FM

Operation

- Step 1. To ensure accuracy, place the scale on a hard flat surface. Make sure there are no objects beneath the scale before weighing.
- Step 2. Gently step on the scale a few times before weighting yourself. This will set the internal mechanism. Adjust the pointer to zero, if necessary at the bottom of scale.
- Step 3. Step on scale with weight equally distributed on both feet, stand still.

Maintaining the Scale:

1. The baked enamel finish, lens and mat can be cleaned with a mild soap and damp cloth. DO NOT USE ABRASIVE OR STRONG CHEMICAL CLEANERS.
2. All bearings and working parts are permanently lubricated. DO NOT USE OIL OR LUBRICANTS.
3. If your scale fails to work properly, do not attempt to dismantle it. This will void your warranty.

Warranty:

One year limited warranty against manufacturers defects.



Medline Industries, Inc.

One Medline Place
Mundelein, IL 60060

Medline United States
1-800-MEDLINE (633-5463)

www.medline.com | info@medline.com

Medline Canada
1-800-396-6996

www.medline.ca | canada@medline.com

Medline México
01-800-831-0898

www.medlinemexico.com | mexico@medline.com

Medline is a registered trademark of Medline Industries, Inc.
© 2014 Medline Industries, Inc.

GPI460-MDR300FM
Revised: 11/26/14