# TRANSFER BENCH

## BANQUITO DE TRANSFERENCIA

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This Product Contains Dry Natural Rubber.  
Este Producto Contiene Hule Natural Seco.

PROTECTION THAT LIVES ON  
**MICROBAN**

MICROBAN is a registered trademark of Microban Products Company.  
MICROBAN es una marca comercial registrada de Microban Products Company.

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MDS86960KDMB, MDS86960KDMBG

# General Warnings

To ensure the safe installation, adjustment, and operation of the Transfer Bench, these instructions **MUST** be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER AND/OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions, and instructions, contact a healthcare professional or dealer before attempting to use this equipment.

## Warnings:

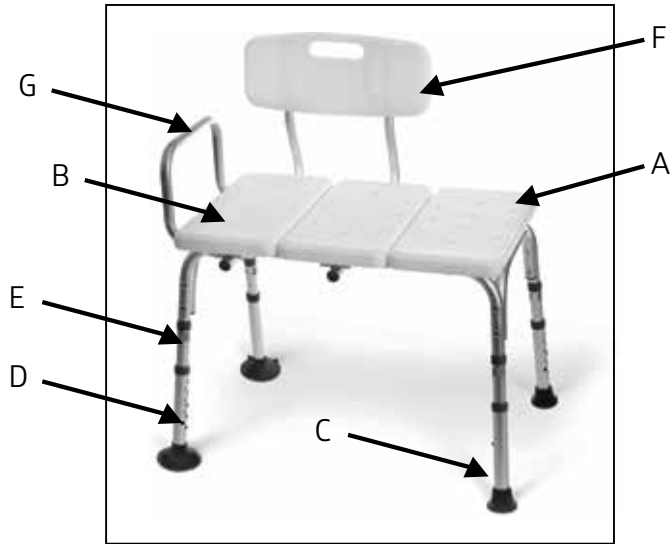
-  Prior to each use check to assure that all screws, joints and fittings on the transfer bench are tight.
  -  **DO NOT** place the transfer bench on uneven surfaces.
  -  **DO NOT** use if wobbly, unstable or not level.
  -  **DO NOT** exceed user weight capacity of 350 lbs. (158 kg)
  -  **DO NOT** stand on the transfer bench.
  -  **DO NOT** use the transfer bench for anything but its intended purpose.
  -  **DO NOT** use the armrest to fully support body weight when using the unit.
- 
- ✓ Excessive twisting or turning while seated on the transfer bench may cause a loss of balance and damage to the product.
  - ✓ Check rubber tips for rips, cracks or wear. If these conditions exist, **DO NOT use the transfer bench and replace the tips immediately.**
  - ✓ Check the bench for cracks or ruptures in the plastic. If you have concerns, contact the manufacturer at 1-800-MEDLINE.
  - ✓ Check that all push buttons are fully protruded from the mainframe through the leg extensions and are properly engaged.
  - ✓ Ensure all four legs are adjusted evenly and are free of any obstructions prior to use.
  - ✓ To release suction, use the pull tabs on the suction cups. Failure to do so will cause damage to the product.

# Transfer Bench Components

**Note:** Before assembly, ensure that all parts and quantities match the parts list. Inspect all components for shipping damage. If any parts are missing or damaged, contact your dealer/retailer for replacements.

- A. Two Panel Section of Seat
- B. One Panel Section of Seat
- C. Legs with Extension (2)
- D. Suction cup legs with extension (2)
- E. E Clamps (4)
- F. Back Rest
- G. Arm Rest

(Sections A, B and F have a pre-assembled frame.)

A

B

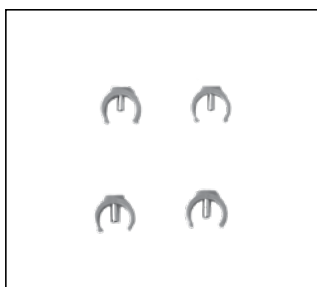
C

D

E

F

G


# Assembly Instructions

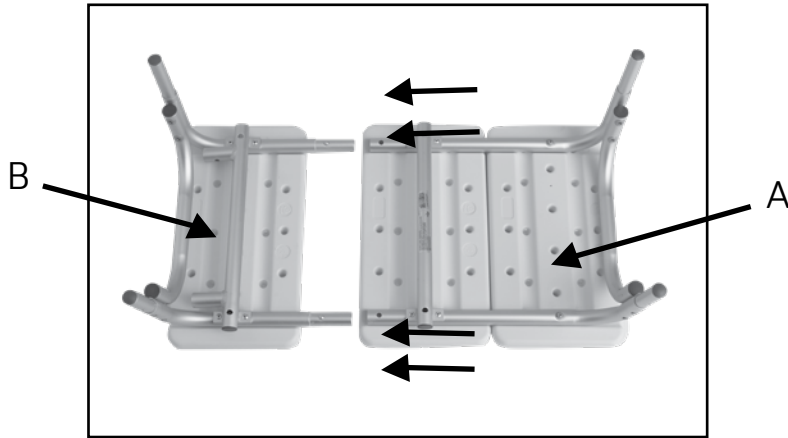Flip the One Panel (B) and the Two Panel (A) Sections of the transfer bench upside down so the aluminum frame is visible.

Insert the One Panel Section into the Two Panel Section (tubular hole). Ensure the push buttons are inserted all the way and are engaged properly. The entire unit will now be referred to as the Three Panel Seat.

Note the four posts at each corner of the Three Panel Seat. Each post features a hole facing outward.


Three Panel Seat


Insert one Leg Extension (C) into the post of the Three Panel Seat. Please ensure the hole in the Leg Extension (C) matches up with the hole in the post of the Three Panel Seat. Repeat for the remaining three posts.

Please note: the Leg Extensions (C) should be on the side of the larger side of the three panel seat. The Suction Cup Leg Extension (D) should be on the side where there is an extra cross bar for the backrest and side arm are attached.

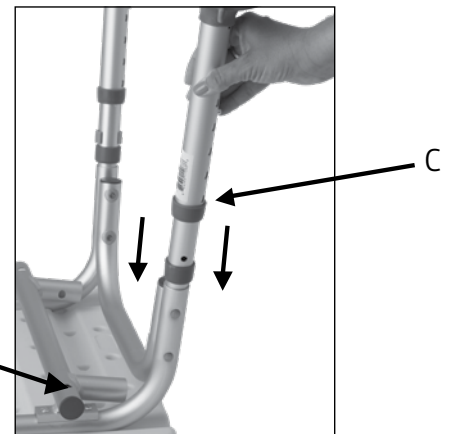Three Panel Seat

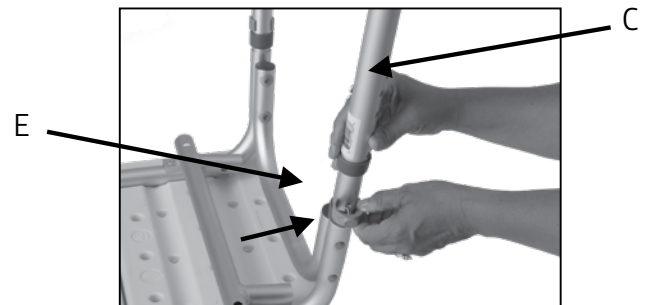
Insert a Clamp (E) into the hole to secure the leg into place. Repeat for the remaining three legs.


With the transfer bench still upside down, insert the Back Rest (F) into the Three Panel Seat. Please ensure the push buttons are securely fastened before proceeding.

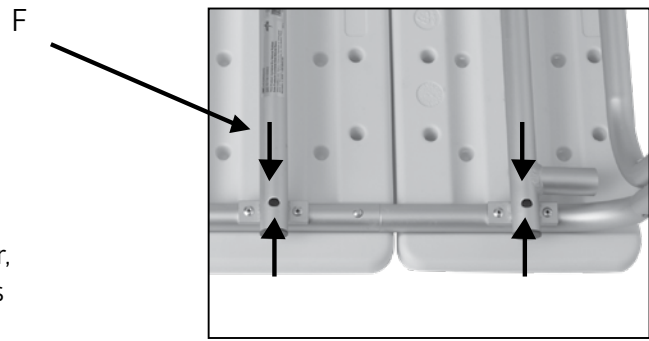**Note:** The back rest can be placed on either side of the of the transfer bench to accommodate the bath tub setup. However, the armrest should always be on the side of the bench that is in the tub.

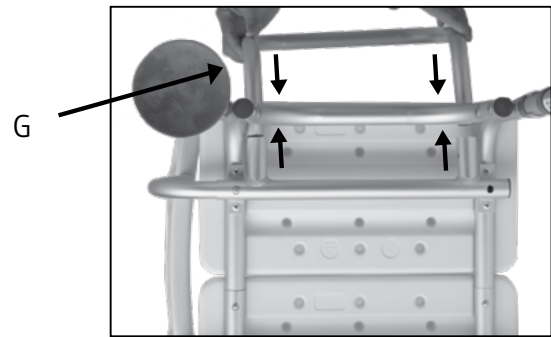With the transfer bench still upside down, insert the Arm Rest (G) into the Three Panel Seat. Please ensure the push buttons are securely fastened before proceeding.

## Adjust the height of the transfer bench

To ensure the tub is not damaged when adjusting the height of the transfer bench, check to be sure the bench fits above the sidewall of the tub.

## Care and Maintenance

Do not use an abrasive cleaner. Use a non-abrasive cleaner with warm water. Periodically inspect the transfer bench for damage or loose components.

Microban protection is built into the transfer bench to help prevent stains and odors caused by bacteria, mold and mildew, for the lifetime of the product. Always clean this product thoroughly after each use.

\*These antimicrobial properties are built-in to protect the product. This product does not protect users or others against bacteria, viruses, germs or other disease organisms.

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# Lifetime Limited Warranty

Your Medline brand product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories; which are subject to normal wear and replacement. Non-durable components are warranted for 6 months. If you have a question about your Medline product or this warranty, please call us at 1-800-MEDLINE. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

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©2020 Medline Industries, Inc. All rights reserved. Medline is a trademark of Medline Industries, Inc. MICROBAN is a registered trademark of Microban Product Company Corp. /© Tous droits réservés, Medline Industries, Inc., 2020. Medline est une marque de commerce de Medline Industries Inc. MICROBAN est une marque déposée de Microban Product Company Corp. /©2020 Medline Industries, Inc. Todos los derechos reservados. Medline es una marca comercial de Medline Industries Inc. MICROBAN es una marca comercial registrada de Microban Product Company Corp.
**Medline Industries, Inc.**