# AUTOMATIC BLOOD PRESSURE MONITOR

USER'S MANUAL

REF MDS5001

# TABLE OF CONTENTS

|   |        |
|---|--------|
| Important Notes                         | 3      |
| Measurement Accuracy                    | 3      |
| Helpful Information                     | 4      |
| Correct Method of Measurement           | 6      |
| Parts Identification                    | 7      |
| Description of Display Marks            | 7      |
| Preparation before Taking a Measurement | 8      |
| Taking a Measurement                    | 9      |
| Setting Date and Time                   | 10     |
| Memory Function                         | 10     |
| Er Indicators / Troubleshooting         | 12, 13 |
| Sensor Calibration Mode                 | 14     |
| Care and Maintenance                    | 14     |
| Specifications                          | 15     |
| Blood Pressure Log                      | 16     |

## IMPORTANT NOTES

1. Only a physician is qualified to interpret changes in your blood pressure. This device is not intended to replace regular medical examinations. It is recommended that your physician review your procedure for using this monitor. Never make adjustments to your medication unless it is the advice of a physician.
2. This Blood Pressure Monitor is intended for use by adults. Children should not use this monitor unless it is under the supervision of an adult.
3. Only use the cuff(s) included with this device or offered as approved accessories. Unapproved cuffs can affect measurement accuracy.
4. Please read the entire instructions carefully before using this Blood Pressure Monitor.
5.  Warning Symbol
6.  Type BF Symbol
7.  Class II Symbol
8.  Keep Dry
9.  Product should be recycled separately from other household waste

## MEASUREMENT ACCURACY

The quality of this device has been verified and conforms to the provisions of the EC council directive 93/42/EEC on medical devices, as well as the EMC directive 89/336/EEC:

- |                     |  |
|---------------------|--|
| <b>EN 1061-1</b>    | Non-invasive blood pressure measuring equipment general requirements   |
| <b>EN 1060-3</b>    | Non-invasive blood pressure measuring equipment supplementary requirements for electro-mechanical blood pressure measuring systems |
| <b>EN 60601-1</b>   | Safety requirements for medical electrical equipment   |
| <b>EN 60601-1-2</b> | Electromagnetic compatibility and safety for medical electrical equipment  |
| <b>EN 14971</b>     | Risk analysis for medical devices  |


## HELPFUL INFORMATION

### What is Blood Pressure?

Blood pressure is the force that blood exerts on the arteries. This force is constantly changing as the heart beats. When the heart contracts, blood pressure reaches its highest value. This is called systolic blood pressure. When the heart relaxes between beats, the value of blood pressure is lower. This is called the diastolic blood pressure. The unit of measure for blood pressure is millimeters of mercury, abbreviated mmHg. For example, an individual's blood pressure may be measured as 120 mmHg (systolic) and 80 mmHg (diastolic). This would be spoken as "120 over 80" and written as "120/80". Remember that blood pressure varies throughout the day. Food intake, smoking, the time of day, stress, level of exercise and many other factors can affect it.

### Typical Daily Blood Pressure Fluctuations

(Example: 35-yr old male)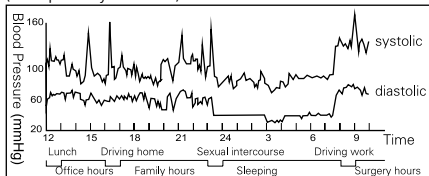


### What is High Blood Pressure?

Hypertension, or high blood pressure, is a condition where an individual's blood pressure remains high over a long period of time. Consult with your physician with any questions or concerns you may have about hypertension.

## Blood Pressure Classification

The World Health Organization (WHO) and the International Society of Hypertension (ISH)

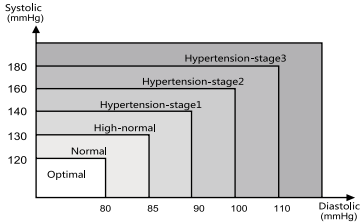## Why is it Important to Measure Your Blood Pressure at Home?

Having your blood pressure taken at the doctor's office may cause you to become nervous, thus artificially raising your reading. Having the ability to take your blood pressure at home makes it easy to record a log of your daily readings. This will help you gain a greater understanding of your blood pressure reading and the factors that affect it. Be sure that you share your information with your physician.

## Hints for Accurate Measurement

1. Relax and try to remain still for 5 to 10 minutes before a measurement.
2. Remove any clothing on the upper arm so that the cuff can be placed directly on the skin. Constriction of the upper arm caused by rolling up a shirt sleeve may cause an inaccurate reading. Avoid this condition by completely removing the garment.
3. Refrain from eating, smoking and drinking (especially alcoholic beverages) before a measurement. These activities can affect your blood pressure.
4. Remember that blood pressure varies continuously throughout the day. Try to take your blood pressure at the same time each day.


5. Do not be concerned with the results of one measurement. Many measurements, recorded over an extended period of time, will provide a better indication of your blood pressure.
6. Relax for 5 to 10 minutes before remeasuring your blood pressure.
7. Because many factors can affect your blood pressure, daily fluctuations of 25 to 50 mmHg are common.

**Please note:** Some individuals with hypertension, diabetes, kidney disorders, arteriosclerosis or poor circulation may see a significant difference in blood pressure readings taken from the wrist as compared to readings from the upper arm. It is recommended that you consult with your physician concerning the use of this monitor.

## CORRECT METHOD OF MEASUREMENT

To obtain the most accurate blood pressure measurement, please follow these important directions:


1. Be seated in a chair with back support.
2. Rest your arm on a table so the cuff is at the same level as your heart.
3. Place both feet on the ground.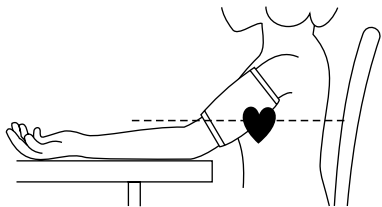


**Important:** The arm cuff must be at the same level as your heart or accurate measurement will not be possible. (Your heart is located slightly below your left armpit.)

# PARTS IDENTIFICATION

## Monitor

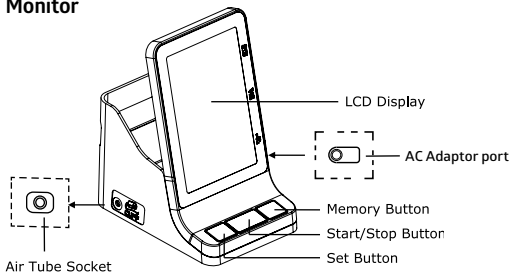## Accessories
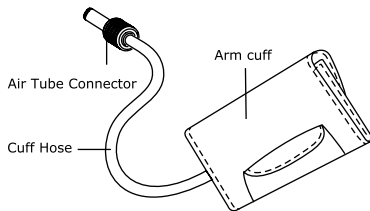
# DESCRIPTION OF DISPLAY MARKS

▲ Inflating

▼ Deflating

Er Measurement Error

♥ Measuring

Battery Display

\*The battery indicator will reset when power resets. Please replace with new batteries when the battery indicator flashes.


Arrhythmia Detection Function

# PREPERATION BEFORE TAKING A MEASUREMENT


## Battery Installation/Replacement

1. Slide the battery cover off in the direction of the arrow.
2. Install or replace the four alkaline “AA” batteries, noting the proper orientation of positive (+) and negative (-) terminals in the battery compartment.
3. Replace the battery cover.
4. If the Low Battery Symbol appears on the display, replace the batteries.
5. Remove the batteries if the device will not be used for an extended period of time.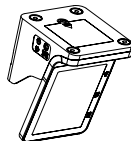


## How to Apply the Arm Cuff


1. Insert the end of the cuff through the D-Ring to make a loop. (Make sure that the hook and loop closures stays outside when it is done.)
2. Wear the arm cuff with the hose downward and line up your brachial artery with the Artery Mark. Pull the end of the cuff until it is wrapped snugly around arm and secure it with the velcro.
3. Adjust the cuff so that the bottom edge is about 1 inch above the elbow on the inside of the arm. Make sure the colored area on the cuff covers the brachial artery.
4. Plug the air tube connector into the monitor.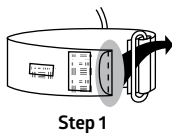


Step 1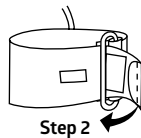

Step 2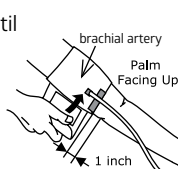

Step 3

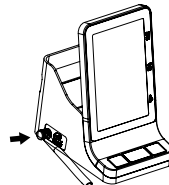Step 4

**Note:** Batteries are hazardous waste.  
Do not dispose with household garbage.

## TAKING A MEASUREMENT

Please read the previous section on proper placement of the cuff and arm position.

1. After the cuff has been properly placed on the arm, please press Set button and select [  ] or [  ] on the upper left hand corner of the LCD screen, and then press the Start | Stop button.
2. After the display reads "00" the cuff will automatically inflate to an appropriate pressure. Then press the Start/Stop button twice.

After inflation, it will slowly deflate to take the measurement. When the heart mark "♥" appears a measurement is in process. The cuff will be quite snug for a short while, this is normal. Try to relax, refrain from talking and remain as still as possible during the measurement.

**Note:** If the device determined that the initial inflation pressure was insufficient, it will re-inflate to a higher pressure.

3. The device will completely deflate when the measurement is finished. The systolic and diastolic pressure and pulse rate will be displayed.

To end a measurement for any reason, just press the Start/Stop button to turn off the unit and release the cuff pressure. If during a measurement the power drops to an insufficient level, the device will end the measurement and display the low battery symbol. Please replace the batteries and repeat the measurement. If the device cannot detect your pulse, it will end the measurement attempt. Please wait a few minutes, make sure that the cuff is positioned properly and try again. The device will automatically shut off 1 minute after a measurement is finished. You can also press Start | Stop button to shut the device off.

## SETTING DATE AND TIME

The device displays time and date on the LCD. They can be set according to the following method:

1. Make sure the machine is powered OFF, then hold the SET button for 5 seconds until digits on the screen start flashing.
2. Press the SET button to switch between month, date, hour and minute.
3. Press the MEM (Memory) button to advance the display selected by one digit at a time.
4. When the unit is powered off, the clock will continue to display on the LCD screen.

## SETTING TALKING FEATURE

For the MDS5001, after the date and time settings are selected, the "SP" character will flash. Press the MEM (Memory) button to move from English, "SP 01", to Spanish, "SP 02", or to off, "SP OF".

## MEMORY FUNCTION

### Memory Input

When a valid measurement is made, the result (systolic, diastolic, pulse rate, date and time) will be stored in the memory automatically as soon as the device is turned off or when another measurement is initiated. The device can store up to 90 readings for each of 2 users, and also compute the average of the latest 3 readings. When more than 90 readings are stored, the oldest ones will be replaced.

### Memory Recall

By pressing Memory button, AVG will be shown in the lower left hand corner. AVG means the average of all recorded readings. Press the Memory button again and 01 will appear in the lower right hand corner, which is your latest reading in the memory. When you press Memory again, 02 will display which is the previous reading. When you press the Memory button repeatedly, the records will be displayed sequentially, along with the corresponding date of the measurement.

## SETTING ALARM CLOCK

1. Make sure the machine is powered OFF, then hold the SET button for 5 seconds until digits on the screen start flashing.
2. Press the SET button to advance the screen from the date, time, and talking feature.
3. The screen will have a small bell located in the upper left hand corner of the screen.
4. Press the MEM (memory) button to change the alarm clock from OFF to ON.
5. Once ON, press the SET button to switch to the time located above "ON" on the lcd screen
6. Press the MEM (memory) button to change the hour digit.
7. Once the hour is set, press the SET button to move to the minute digits.
8. Press the MEM (memory) button to change the digits.
9. Once the alarm is set and turned on, a small bell icon will appear in the top left hand corner of the screen when the unit is powered off.

## Memory Erase

Press memory button and hold it down more than 3 seconds.  
You will erase all of the records of readings for the selected user.

## ER INDICATORS

### Error Indicators

### Possible Sources of Errors

Er  
U

**Indication:** Incomplete inflation

**Correction:** Check that the cuff has been firmly connected with the device.  
If the same Er message is displayed, contact Product Support.

Er  
H

**Indication:** Cuff over-inflated and released automatically

**Correction:** Take a break for 5-10 minutes. Then reposition the cuff and measure again.

Er  
I

**Indication:** Could not obtain the pulse rate

**Correction:** Wait 5-10 minutes. Then reposition the cuff and measure again.

Er  
2

**Indication:** Strong electromagnetic interference (mobile phone or computer)

**Correction:** Relax and take a break for 5-10 minutes. Then measure again.

Er  
3

**Indication:** Irregular pulse rate or blood pressure

**Correction:** Relax and take a break for 5-10 minutes. Then measure again.

## TROUBLESHOOTING

| <b>Problem</b>                                  | <b>Recommended Method</b>   |
|---|---|
| No display                                      | Make sure that the batteries are installed in the proper direction. (Polarity matches to the indication in the battery case.)   |
| No measurement occurs                           | Check that the cuff is positioned properly, then repeat the measurement procedure.  |
| Blood pressure readings are too high or too low | <ol style="list-style-type: none"><li>1. Check if your arm is positioned properly.</li><li>2. Position your arm on a table so the cuff is at the same level as your heart.</li><li>3. Remain seated during the entire measurement period.</li><li>4. Refrain from hand and body movements during measurement.</li></ol> |
| Blood pressure readings are variable            | Your blood pressure can fluctuate considerably throughout the day, all of the following factors can influence your blood pressure: emotional state, daily activities/exercise, smoking, drinking alcoholic beverages, eating, taking certain medications  |

## SENSOR CALIBRATION MODE

The blood pressure device should NOT attempt to be calibrated. The unit arrives pre-calibrated by the vendor. Should any issues arise with the device, please contact Product Support.

### **mmHg and kPa Transferring Instruction**

If you want to change the measurement units, power off the device and then press the Start/Stop button for 10 seconds. Next press the "Memory" button and select mmHg or Kpa.

## CARE AND MAINTENANCE

- Keep the device away from high temperatures, high humidity and direct sunlight.
- Keep sharp objects away from the cuff and tubing.
- Do not press the Start/Stop button without the cuff around your arm to avoid excess pressure build up that may damage the cuff.
- Use the device at a sufficient distance from devices with strong electrical fields, such as television, microwave ovens, X-ray equipment, etc.
- Do not subject the device to strong impact or drop it on the floor.
- Do not disassemble or modify the device or the cuff.
- Remove the batteries when the device is to be stored for extended periods of time.

**Note:** Removing the batteries will erase all readings in memory.

Use only a soft dry cloth to clean the unit. Do not use solvents or other petroleum-based cleaners.

## SPECIFICATIONS

|                                       |  |
|---------------------------------------|--|
| Manufacturer Model:                   | BA-813   |
| Medline Item Numbers:                 | MDS5001  |
| Measuring Method:                     | Oscillometric  |
| Memory Function:                      | Storage and recall for 90 measurements per user (up to 2 users)                                      |
| Display:                              | Backlight LCD Display  |
| Measuring Range:                      | Pressure: 0-280 mmHg Pulse: 40-199/minute  |
| Sensor Accuracy:                      | Pressure: ±3mmHg Pulse: ±5%  |
| Inflation System:                     | Electro-pneumatic pump   |
| Arm Size Ranges:                      | 22-42 cm (8.6-16.5 inches)   |
| Batteries:                            | 1.5V alkaline (LR6/AA)X4   |
| Automatic Power-off:                  | Approx. 1 minute after measurement   |
| Reference Method for Clinical Trials: | Auscultatory measurement   |
| Weight:                               | Approximately 465g(without batteries)  |
| Operating Condition:                  | 41°F ~ 104°F (5°C ~ 40°C) 10%~90%RH  |
| Storage and Transport Condition:      | -4°F ~ 149°F (-20°C ~ 65°C) 10%~95%RH  |
| Standard Cited:                       | EN 1060-1                      EN 60601-1-2<br>EN 1060-3                      EN 14971<br>EN 60601-1 |

# BLOOD PRESSURE LOG

| Date           | 1/1  | 1/1   | 1/1   |     |  |  |  |  |  |
|----------------|------|-------|-------|-----|--|--|--|--|--|
| Time           | 7:00 | 13:30 | 20:00 |     |  |  |  |  |  |
| mmHg           | 240  |       |       |     |  |  |  |  |  |
|                | 220  |       |       |     |  |  |  |  |  |
|                | 200  |       |       |     |  |  |  |  |  |
|                | 180  |       |       |     |  |  |  |  |  |
|                | 160  |       |       |     |  |  |  |  |  |
|                | 140  |       |       |     |  |  |  |  |  |
|                | 120  | 128   | 134   | 123 |  |  |  |  |  |
|                | 100  |       |       |     |  |  |  |  |  |
|                | 80   | 84    | 90    | 76  |  |  |  |  |  |
|                | 60   |       |       |     |  |  |  |  |  |
| Pulse          | 70   | 73    | 69    |     |  |  |  |  |  |
| Body Condition |      |       |       |     |  |  |  |  |  |

**www.medline.com** ©2017 Medline Industries, Inc.  
Manufactured in China for Medline Industries, Inc.,  
Northfield, IL 60093 USA. **1-800-MEDLINE** RL17CN1

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Fabricado en China para Medline Industries, Inc.,